



SuperDART[®]

ONE PASST[™]

Novel Three Stage Single Use Endoscope Channel Cleaning Device

- Channels cleaned in just *ONE PASST[™]*
- Validated Cleaning Efficacy for *ONE PASST[™]*
- Easy Pull Through action




ROBUST LEADING BRUSH



SQUEEGEE MULTI DISK WIPER



FINE BRISTLED TAIL BRUSH

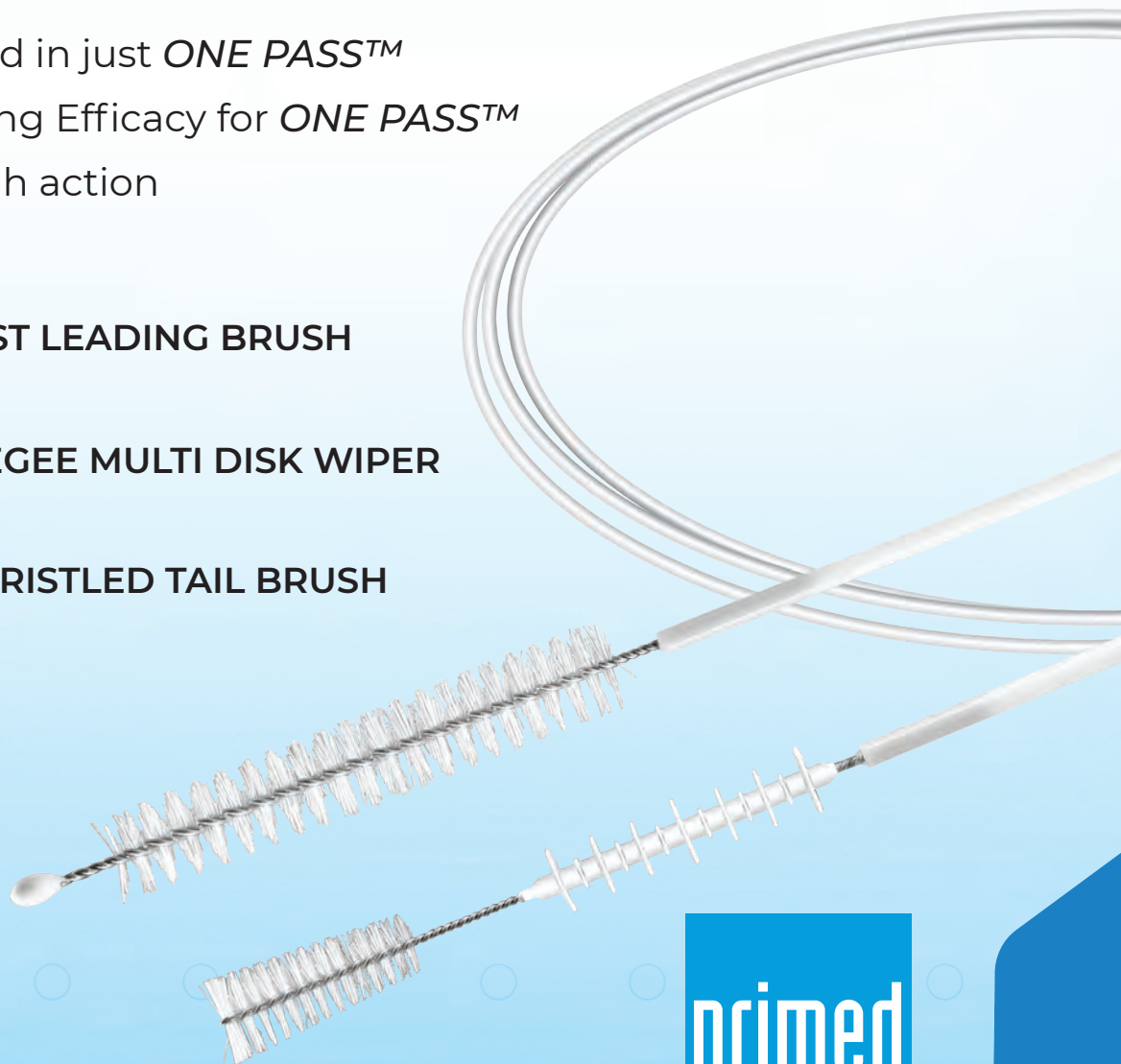

primedENDO.com




Pull Through Three Stage **SuperDART®** is the most sophisticated endoscope channel cleaning device on the market today.

SuperDART® ONE PASS™ achieves validated **99.997 - 100% Soil Removal** from contaminated channels in a single swipe¹

SuperDART® ONE PASS™ Advantage	Standard Brush
· Time savings with a single swipe	· Multiple passes required
· Operators' fatigue Reduction	· Repetitive strenuous work
· Objective High Level of Cleanliness	· Subjective and uncertain Cleanliness

This is how the **SuperDART®** works:

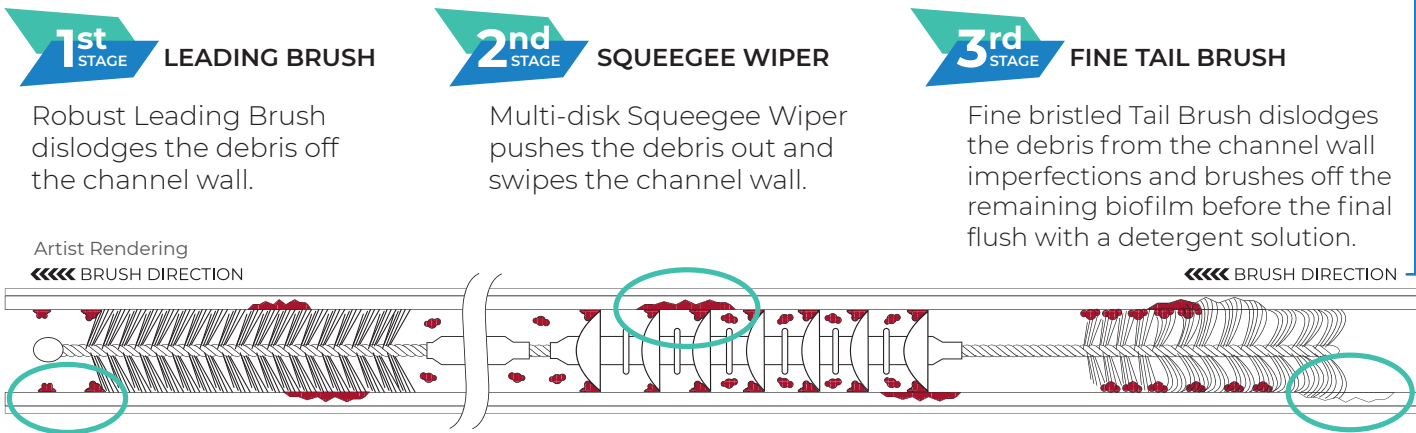

1st STAGE LEADING BRUSH
Robust Leading Brush dislodges the debris off the channel wall.


2nd STAGE SQUEEGEE WIPER
Multi-disk Squeegee Wiper pushes the debris out and swipes the channel wall.

3rd STAGE FINE TAIL BRUSH
Fine bristled Tail Brush dislodges the debris from the channel wall imperfections and brushes off the remaining biofilm before the final flush with a detergent solution.

Artist Rendering
←←← BRUSH DIRECTION

←←← BRUSH DIRECTION

Product Image

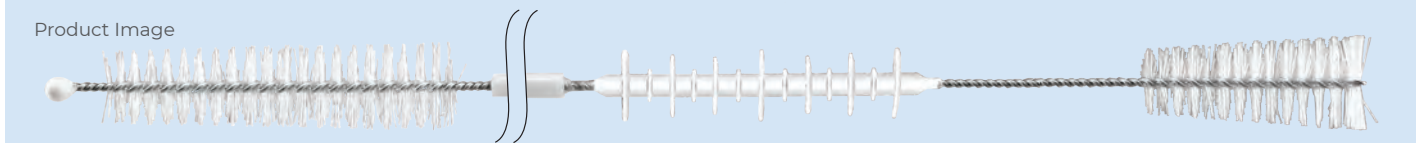

PART NUMBER	DESCRIPTION	SHEATH DIAMETER (MM)	LENGTH (CM)	BRUSHES DIAMETER (MM)	SQUEEGEE WIPER DIAMETER (MM)	CHANNEL SIZE (MM)	UNITS/BOX
9292	SuperDART® 3 Cleaning Stages Tool	1.7	250	Leading Brush: 5-7 Tail Brush: 4 - 6.3	2.5 - 5.2	2.8 - 4.2	25
9188	SuperDART® 3 Cleaning Stages Tool with Valve/Control Head Brush	1.7	250	Leading Brush: 5-7 Tail Brush: 4 - 6.3 Valve/Control Head Brush: 10	2.5 - 5.2 (Squeegee Wiper)	2.8 - 4.2	25

• Only use for the specified channel sizes • Always introduce the Leading Brush into the channels first • Always use wet, both the channel & the SuperDART®



Manufactured by:
PriMed Instruments Inc.
1080 Tristar Drive, Unit #14
Mississauga, ON L5T 1P1
Canada
☎ 905-565-0565
☎ 905-565-0566

TO ORDER CONTACT
✉ sales@primedENDO.com
☎ Toll Free: 877-565-0565
☎ Toll Free: 877-565-0566

